

Swell Spots™

MANUFACTURED BY SOLARIS



user guide

Your Swell Spot

Swell Spots are professionally designed treatment pads that tuck into garments or under bandaging to address particularly troublesome areas of edema, fill contours or diffuse pressure points and break up fibrosis. Please consult with your medical professional for individual usage.



Care Instructions & Helpful Hints

- Use the grey fabric against your skin.
- Wash weekly or more frequently as needed.
- Machine wash with mild detergent on permanent press.
- Double rinse to remove any soap residue.
- Never use additives, such as bleach or fabric softener.
- Tumble dry on permanent press to re-fluff the foam.
- Place clean tennis balls in dryer to fluff up the foam.
- Stop use if rash, numbness or skin color changes are noticed.



Materials



64% Nylon, 23% Polyester,
14% Lycra, Medical grade
polyurethane foam

Not made with natural
rubber latex.
Made in the USA

Returns & Warranty

Solaris will replace defective products within two weeks of the purchase date. Please contact Solaris directly. Solaris does not accept returns otherwise. In an effort to keep prices low, Solaris does not warrant Swell Spots products.

Indications

- Lymphedema
- Acute/Chronic Edema
- Hypertrophic Scarring
- Indurate Tissue

Contraindications

- Arterial insufficiency or degeneration
- Untreated congestive heart failure
- Systemic infection
- Impaired sensation
- Cognitive impairment
- Allergies or sensitivities to materials



6737 W. Washington St, Ste 3260
West Allis, Wisconsin 53214 USA

T: (855) 892-4140 | F: (414) 892-4150 | E: info@solarismed.com
www.solarismed.com

Toomac Solutions

PO Box 36190, Northcote, Auckland 0748, NZ
32C Poland Road, Glenfield, Auckland 0627, NZ
T: 09 443 5347 F: 09 443 5345 E: juzo@toomac.co.nz



Emergo Europe
Molenstraat 15
2513 BH, The Hague, The Netherlands

REV 1/14