



JuzoPro

Manu Xtec Palmar / Palmar Xtec Rhizo






Handgelenkorthese · Wrist orthosis

Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Bruksanvisning · Bruksanvisning · Käyttöohje · Bruksanvisning · Οδηγίες χρήσης · Kullanım talimatları · Upute za uporabu · Instrukcja użyci · Инструкція по експлуатації · Návod na používanie · Navodila za uporabo · تعليمات الاستخدام

JuzoPro Manu Xtec Palmar:

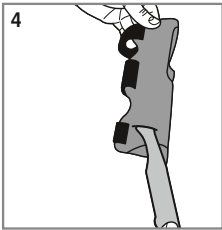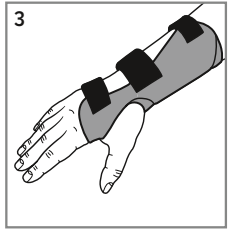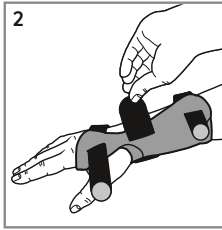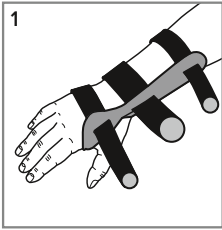JuzoPro Palmar Xtec Rhizo:

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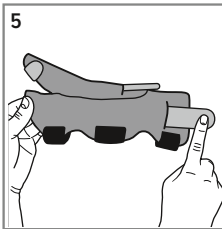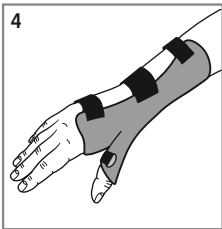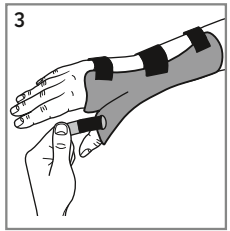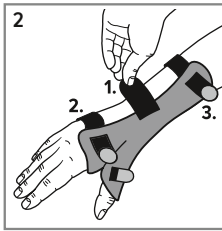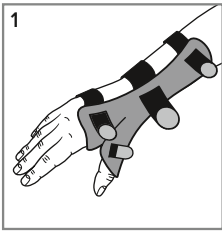EN

Purpose

The **JuzoPro Manu Xtec Palmar**, made out of inelastic materials with adjustable, partly removeable reinforcing elements, immobilises the wrist in two movement directions (ulna/radial and dorsal/volar).

The **JuzoPro Palmar Xtec Rhizo**, made of inelastic materials with adjustable, removable reinforcing elements, stabilises the wrist and the metacarpophalangeal joint of the thumb.

Here is how to put on your

JuzoPro Manu Xtec Palmar correctly:

1. Open all three velcro fasteners on the top and pull the straps through the buckles as far as the hook and loop pad (do not open fully!). Now slip your hand into the orthosis. As you do so, insert your thumb into the opening provided (Fig. 1).
2. First do up the middle velcro fastener, then the strap at the back of the hand, then the fastener on the forearm (Fig. 2).

The orthosis is fitted correctly if it securely encloses the wrist (Fig. 3). Please ensure the straps do not have a constrictive effect and do not exert a too strong (painful) pressure on the wrist. The orthosis can be readjusted at any time by repeating step 2 above. The stabilisation splints can be taken out and put back in again via the slit on the side of the pocket (Fig. 4)

Here is how to put on your

JuzoPro Palmar Xtec Rhizo correctly:

1. Open all three touch fasteners on the top, as well as the touch fastener around the thumb stub, and pull the straps through the eyes up to the hook part (do not open them completely!). Now slide your hand into the orthosis. The thumb must be guided through the dedicated opening (Fig. 1).
2. Close the middle touch fastener first, then the strap at the back of the hand. Next, the closure on the forearm and finally, the closure around the thumb (Fig. 2 and 3).

Correct fit of the orthosis is guaranteed if it firmly encloses your wrist (Fig. 4). Please make sure the straps do not constrict and do not exert excessive (painful) pressure on the wrist. You can re-adjust the orthosis at any time by repeating Step 2. The stabilisation stays can be removed, and also reinserted, via an opening in the pocket (Fig. 5).

Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff in a medical products supply store. The effectiveness of the orthosis can only be guaranteed if it is applied correctly. The orthosis is largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations

may negatively impact on the durability of the fabric. We therefore recommend that it gets examined regularly by the medical products supplier.

Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons and because the orthosis is individually tailored, it is not suitable for treating more than one patient.

Washing and care instructions

Please observe the care instructions on the inside of the product. Please remove the stabilising splints before washing, as illustrated in Fig. 4/5. Wash the orthosis separately the first time (possibility of bleeding). Wash the orthosis and strap by hand at approx. 30 °C using a laundry detergent for delicate fabrics. We recommend using the mild Juzo Special Detergent for this. Afterwards, please rinse it out well and allow it to dry at room temperature. To reduce the drying time, you can place the orthosis on a thick terry towel and pat it dry. Do not leave it in the towel and do not place it on a heater or in the sun to dry. The orthosis is not suitable for tumble drying. Please re-insert the stabilisation rods into the pockets only when the orthosis material has dried completely. This Juzo product must not be cleaned chemically.

Materials used

For full details, please refer to the label on the inside of the product.

Notes for the orthopaedic technician

The integrated stabilising splints have been anatomically preshaped and can be reshaped if necessary.

Storage information and usage duration

Store in a dry place and protect from sunlight. The medical retailer or doctor prescribing the orthosis will advise on exactly how long it can be used for.

Indications

JuzoPro Manu Xtec Palmar

Moderate to severe sprains and strains, tendon/joint injuries, moderate to severe tenosynovitis (inflammation of a tendon and its sheath), carpal tunnel syndrome, rheumatoid arthritis (chronic irritation), osteoarthritis, post-traumatic/postoperative immobilisation (substitute for plaster cast)

JuzoPro Palmar Xtec Rhizo

Postoperative, pre-operative, post-traumatic, severe rhizarthrosis, carpal tunnel syndrome (CTS night resting), tenosynovitis (inflammation of a tendon and its sheath), De Quervain's tenosynovitis (gamer's thumb)

Contraindications

If the following symptoms exist, the orthosis should only be worn after consultation with the physician: Unstable fractures in the area of the hand, lymph drainage disorders, arterial flow impairment,

post-traumatic conditions, skin disorders or skin irritations; Wounds in the treatment area are to be covered with sterile dressings.

The Julius Zorn Group assumes no liability as a result of any contraindicated use of this product.

Side effects

There are no known side effects when using this product correctly. However, if you notice any adverse changes (e.g. irritation of the skin) during the prescribed use, please consult your doctor or medical retailer immediately. If you know that you are allergic to one or more of the product ingredients, please consult your doctor before using the product. If your symptoms worsen during the time you are wearing the product, please take off the orthosis and consult a medical specialist immediately. The manufacturer does not accept any liability for damage/injuries resulting from improper use or misuse.

In the event of complaints related to this product, such as damage to the material or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

As a rule, the product is to be destroyed via the conventional disposal route after the end of use. Please observe the applicable local/national regulations.

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Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany · juzo.com