

On + Off




The multifunctional application aid for donning and doffing medical compression stockings and tights with an open toe

Thanks to its unique sliding properties, Arion Sim-Slide® makes the application of medical stockings easier, thus reducing daily stress associated with donning and doffing compression garments. A study* shows that, when using Arion application aids, medical compression wearers achieve greater satisfaction with successfully putting on and taking off their compression garments. This contributes to their positive user experience, thus improving therapy compliance and optimizing therapy results.

*K. Sippel, B. Seifert, J. Hafner, 2014 - Elderly people with severe chronic venous insufficiency

Before you start:

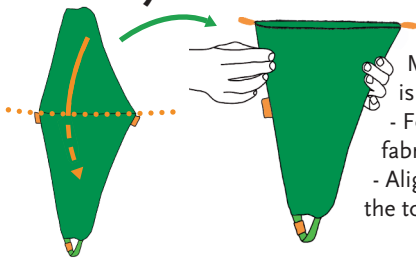


Make sure the product is ready for use:

- Fold back the extra fabric into the „green bag“
- Align the fold line with the top of the orange labels









Important! Don't push the foot to the very end of the Arion Sim-Slide®, leave 2 or 3 fingers space in front of the toes.









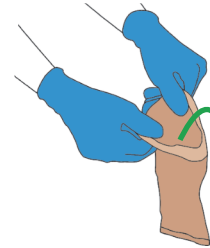


In case the compression garment has a silicon border, make sure to turn it outwards

Donning instructions

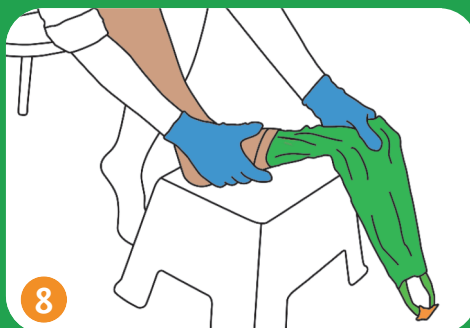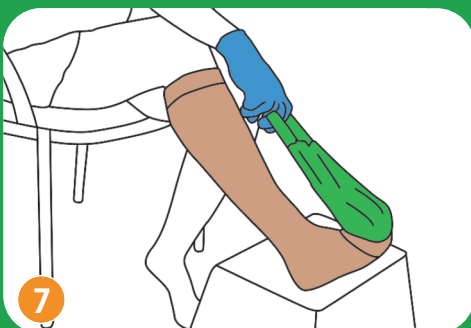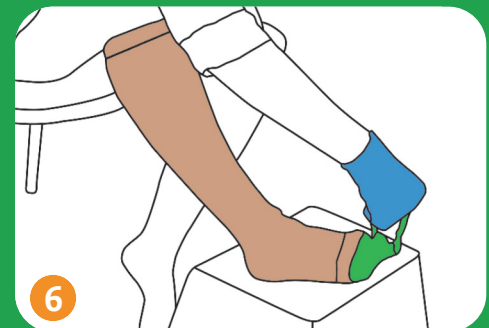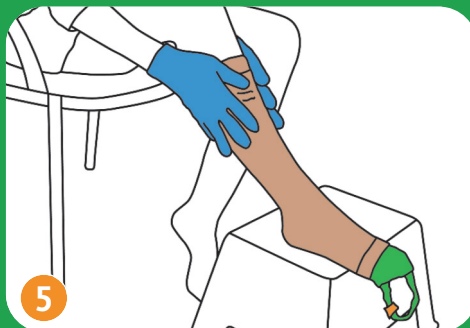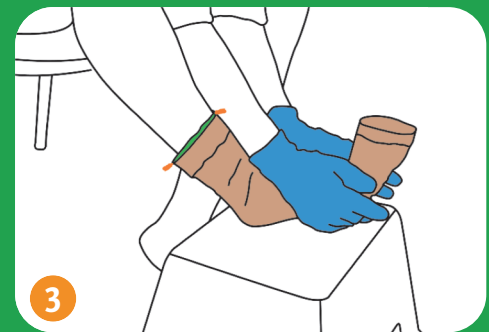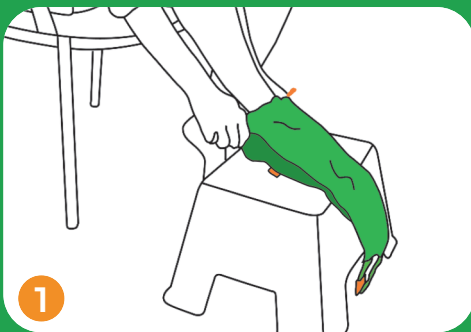

DoFFing instructions

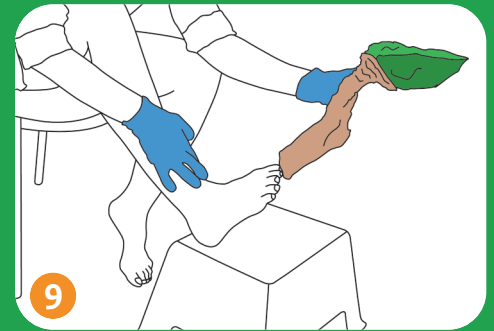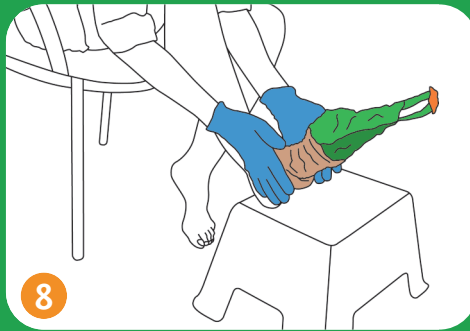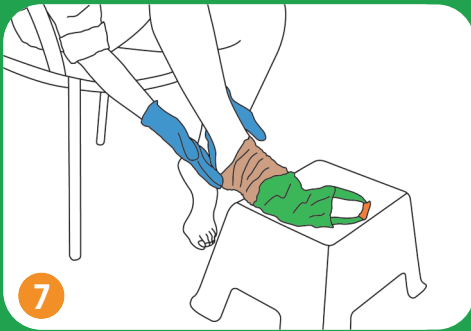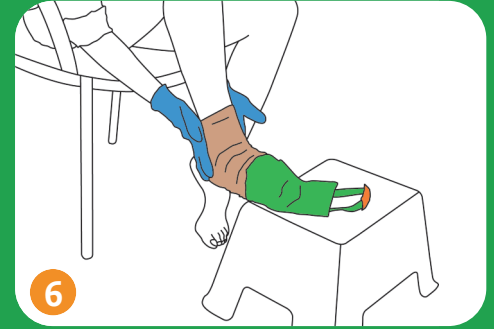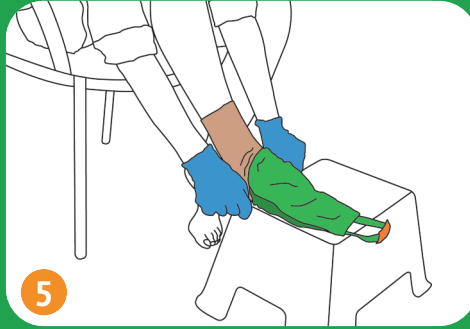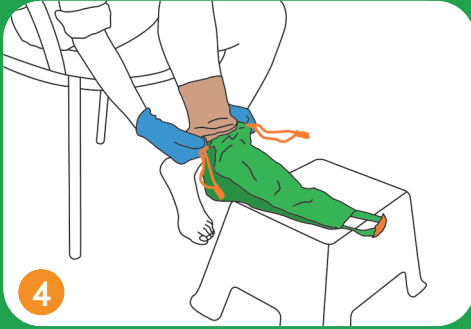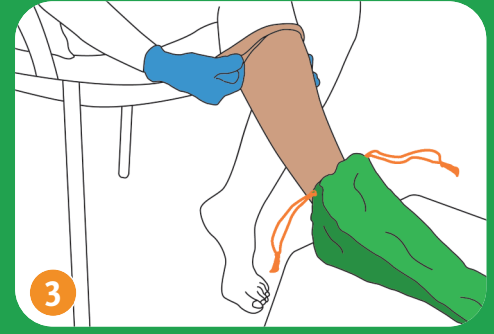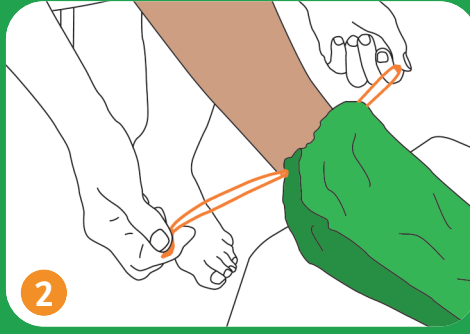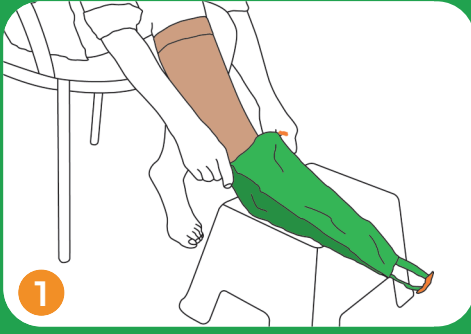

Compression therapy made easier

Arion donning and doFFing aids for medical compression garments

Easy to Use

- Minimize friction between compression garment and skin
- Reduce physical strain for both compression wearers and caregivers
- Promote independence of compression wearers

Safe to Apply

- Reduce risk of skin damage ; can be especially beneficial in case of high skin sensitivity or leg ulcers
- Contribute to a longer garment lifetime by preventing damage from fingers and nails
- Help maintaining the original compression longer and avoiding garments from overstretching and distortion

Simple Solutions

- Application aids for all types of compression garments: stockings, tights, arm sleeves, and gloves
- Designed for different shapes and sizes of limbs
- Support the concordance to compression therapy

Arion Sim-Slide®

SIZES

Art. Nr.	Size	Shoe Size EU	Shoe Size (UK)	
			Woman	Man
EoSLo1	Small	≤ 34 - 36	≤ 1,5-3,5	≤ 2,5-4
EoSLo2	Medium	37 - 38,5	4 - 5,5	4,5 - 6
EoSLo3	Large	39 - 41	6 - 7	6,5 - 7,5
EoSLo4	X-Large	≥ 42	≥ 7,5	≥ 8

Should you need another size, Arion can customize the product according to your needs.