

FASCIQ®


- ✓ No. 1 brand for Fascia Tools
- ✓ Extensive range of Fascia release Tools
- ✓ Recommended by leading R&D institutes



WWW.FASCIQ.COM

@fasciq_official

@fasciq

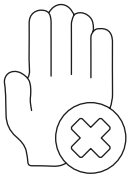

CONTRAINDICATION

Do not cup over arteries or use cups on fresh scars, blemishes, sunburned or damaged skin, rosacea, eczema or other skin inflammations. Consult your health care provider prior to use if you use dermal-fillers or have had any surgery.

If there is an area that is still bruised from a previous cupping treatment, **DO NOT** cup over the same area, if it is not completely healed.

Information provided in this guide is general, individual results can and will vary.



FASCIQ®

SMALL AND LARGE CUP



MAY HELP IMPROVE



athletic performance



circulation



mobility



flexibility

Large cup



Small cup



CARE

- Wash in warm, soapy water.
- Do not place in dishwasher.
- Do not freeze, heat or boil cups.

MATERIAL

- Highest quality Silicone Rubber
- The best developed elasticity and hardness silicon
- BPA Free

SPECIAL GRIP EDITION

- Easy to hold and turn around
- Optimized shape, silky smooth surface,
- Soft and comfortable
- Ergonomic
- Easy to clean
- Newly developed ideal handle
- Very powerful vacuum effect
- Extremely long life

The silicone rubber is not tested on animals, it is produced safely and without toxic substances.



HOW TO

Step 1

Wash the skin thoroughly.

Step 2

Apply any cream or oil of your choice to insure sufficiently lubricated skin, this will prevent any friction when sliding the cup(s) over the skin.

Step 3

(Refer to the “**Application**” illustration on the right).


APPLICATION



Remove air from the inside of the cup by squeezing the massage cup in the middle.



Place the cup on the chosen skin area and release the cup to create a vacuum.

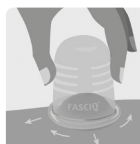

Very gently move the cup across the skin in either a straight line, circle or zigzag.



To remove the cup from the skin, push the middle of the cup before pulling it away from the skin.

FASCIQ® CREAM



To ensure that your cups slide effortlessly over the skin without any friction, use the **FASCIQ® Cream** (100 ml).

Recommended to use in combination with the **FASCIQ® Small and Large Cup**.

FASCIQ® CUPPING SETS



FASCIQ® SILICONE CUPPING SET
SMALL & LARGE CUPS



FASCIQ® SILICONE CUPPING SET
TWO SMALL CUPS



FASCIQ® SILICONE VACUUM CUP
LARGE



FASCIQ® SPORTS CUPPING SET - TRIGGER POINT CUPS



FASCIQ® FACIAL CUPPING SET



FASCIQ® CUPPING SET OF 4 CUPS

