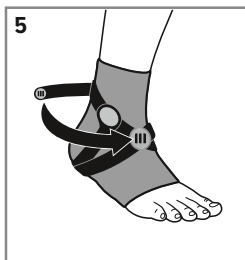
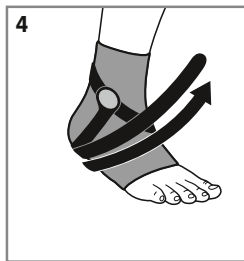
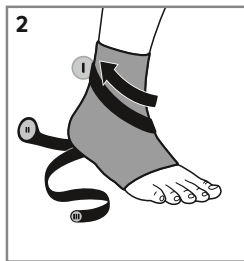
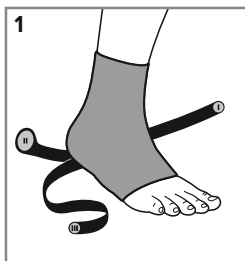
JuzoFlex

Malleo Xtra Strong

Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning · Käyttöohje · Bruksanvisning · Οδηγίες χρήσης · Kullanım talimatları · Upute za uporabu · Instrukcja użyci · Инструкция по эксплуатации · تعليمات الاستخدام · Návod na používanie · Navodila za uporabo

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Purpose

The JuzoFlex Malleo Xtra Strong provides anatomical guidance for the affected ankle, thus relieving it during movement. Medical compressive active supports have a proprioceptive effect. They improve self-perception and can compensate for muscle imbalances. They reduce pain, support the removal of fluids from the tissue, thanks to their rheological effect, and provide relief during movement.

Here's how to put on your JuzoFlex Malleo Xtra Strong correctly:

1. Open all the straps and pull it over your ankle so that strap I is located on the inside, and straps II and III on the outside of your foot (Fig. 1).
2. Now run strap I diagonally across your foot arch and attach it to the velcro area provided. Please ensure that your foot remains at a 90° angle to your calf while doing so (Fig. 2).
3. Now attach strap II under tension to strap I on the outside (Fig. 3) and then run strap III diagonally, in the opposite direction to strap I, around the ankle, over strap II and then fasten it onto strap I (Figs. 4 + 5).

The ankle support is fitted correctly if it stabilizes the ankle. Please ensure the straps do not have a constrictive effect and do not exert a too strong (painful) pressure on the ankle. The straps can be readjusted at any time by repeating steps 2 and 3 above. Please bear in mind that rough, unfiled fingernails, toenails, calluses and shoe linings can cause damage.

Please note:

Only wear your support when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the support can only be guaranteed if it is applied correctly. Although your support is largely resistant to oils, ointments, skin moisture and other environmental influences, the fabric as well as worked in pressure pads can be adversely affected by certain combinations of these factors. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the support yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the support is not suitable for treating more than one patient.

Washing and care instructions

Please note the care instructions on the textile label sewn into your compression garment. Wash the supports separately the first time (colour may run). Your support should be washed every day on a gentle or delicate wash cycle (30 °C). Tip: A mesh laundry bag provides additional protection for the knitted fabric. Please do not use fabric softener! When hand washing, rinse well and do not wring. We recommend

the gentle Juzo Special Detergent. You can reduce drying time by placing the supports on a thick towel after washing, tightly rolling the towel up and firmly pressing out excess moisture. Then hang the support up loosely. Do not leave the product in the towel and do not dry it on a radiator or in the sun. Juzo supports are not suitable for dry cleaning. The support is not suitable for tumble drying.

Material composition

Details are provided on the label sewn into your ankle support.

Storage information and shelf life

Store in a dry place and protect from sunlight. Supports have a maximum useful life of 42 months. The useful life of the medical product is printed on the box with an hourglass symbol. The medical retailer or doctor prescribing the support will advise on exactly how long it can be used for. Body measurements may change based on the patient's condition and individual circumstances. We therefore recommend that medical specialists check body measurements regularly.

Indications

early functional therapy of distortions of the upper and lower ankle joint, chronic, post-traumatic or post-operative soft tissue irritation, injuries of the capsular ligament of the ankle joint, chronic lateral and medial collateral ligament instabilities, supination prevention, after a plaster cast

Contraindications

If you have any of the following conditions, you should only wear the support after consulting with your doctor: Danger of thrombosis, high-grade varicose veins, lymph drainage disorders and non-specific soft tissue swelling outside the area of application, sensory disturbances and impaired circulation in parts of the body to be treated (e.g. in the case of diabetes mellitus), skin conditions, skin irritation and injuries in the area to be treated; open wounds in the treatment area must be covered with sterile material. Liability for failure to comply with the contraindications cannot be accepted by the Julius Zorn GmbH Group.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your symptoms worsen during the time you are wearing the product, please take off the support and consult a medical specialist immediately. The manufacturer is not liable for damages/injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly.

Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

As a rule, the product is to be destroyed via the conventional disposal route after the end of use. Please observe the applicable local / national regulations.

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