Juzo

JuzoFlex

Patella Xtra

Patellasehnenbandage

Patella tendon support

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Purpose

The JuzoFlex Patella Xtra textile patellar strap, which features a pad, exerts pressure on the patellar tendon to alter the patellar mechanics and kinematics.

How to put on your JuzoFlex Patella Xtra correctly:

- 1.** Pull the patellar tendon strap over your forefoot and ankle and up to the right place. Alternatively, you can open up one side of the touch fastener completely and put on the strap as described in step 2 (positioning).
- 2.** You can then position the patellar tendon strap directly below the patella (kneecap).
- 3.** You can adjust the patellar tendon strap by simply pulling it with the help of the cords.

The correct fit of your support is guaranteed when it firmly surrounds the knee without applying too much (painful) pressure.

Please note:

Wear your knee support only per your physician's prescription. Knee supports should only be dispensed by trained personnel at a medical specialty store. As therapy progresses or new symptoms appear, it will be necessary to take new measurements and possibly to produce a new knee support. Although your knee support is largely resistant to oils, ointments, skin moisture and other environmental influences, the fabric as well as a possibly worked in pressure pad can be adversely affected by certain combinations of these factors. It is therefore recommended to have your knee support checked regularly by your medical specialist retailer. If your knee support becomes damaged please consult your specialty retailer. Do not repair it yourself as this could impair the quality and effectiveness. For hygienic reasons, the support is not suitable for treating more than one patient.

Washing and care instructions

Wash the product by hand and do not use any chemical detergents. Then let it dry thoroughly. Touch fasteners should be closed.

Material composition

Please see the information printed on your support.

Storage information and shelf life

Store in a dry place and protect from sunlight. Supports have a maximum shelf life of 42 months. The usage period of the medical product is printed on the box with an hourglass symbol. Body measurements may change based on the patient's condition and individual circumstances. We therefore recommend that medical specialists check body measurements regularly.

Indications

- Patellofemoral osteoarthritis
- Patella tip syndrome (jumper's knee)
- Second phase after patellar stabilisation surgery
- Pain at the front of the knee
- Chondromalacia patellae

Contraindications

If the following symptoms exist, the support should only be worn after consultation with the physician:

Arterial insufficiency, lymph drainage disorders, post-thrombotic syndromes, skin disorders or skin irritation; open wounds in the treated body area should be kept sterile.

The Julius Zorn GmbH Group assumes no liability as a result of any contraindicated use of this product.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your medical condition deteriorates while the use of our products, please go to your physician immediately. The manufacturer is not liable for damages / injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

Your support can be disposed of through normal waste channels. No special waste disposal requirements apply.

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Pri neupoštevanju kontraindikacij skupina Julius Zorn GmbH ne prevzema nikakršne odgovornosti.
Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany