JuzoPro



## Malleo Xtec Light

Sprunggelenkorthese · Ankle orthosis

---

Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning · Käyttöohje · Bruksanvisning · Οδηγίες χρήσης · Kullanım talimatları · Upute za uporabu · Instrukcja użycia · Инструкција по експлуатацији · Návod na používanie · Navodila za uporabo · تعليمات الاستخدام

---

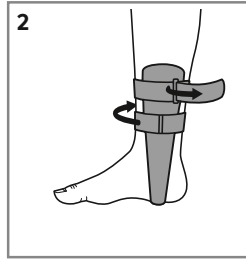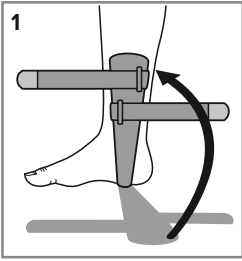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EN

## Intended purpose

The JuzoPro Malleo Xtec Light stabilises the ankle joint with two rigid padded elements that are joined together in a way that allows for flexibility and adaptability and lie on the sides of the joint. It limits the pronation and supination movement, i.e. the inward or outward twisting.

## This is how to put on your orthosis

### JuzoPro Malleo Xtec Light correctly:

Put on your orthosis while sitting down to prevent injuries or falling.

1. Open the two straps on the orthosis shells, fold open the shells and lay the orthosis down on the floor in front of you. Now place your heel on the piece connecting the orthosis shells. As you do this your foot should be at a 90° angle to your lower leg. Now fold both shells around your ankle so that your anklebones rest in the centre of the corresponding protrusions provided.

2. Pass the upper orthosis strap through the buckle and around your leg, then fasten it. Repeat the previous step for the lower orthosis strap.

The correct fit of the orthosis is assured when it encloses the ankle tightly. Please ensure the straps do not have a constrictive effect and do not exert a too strong (painful) pressure.

You can re-adjust the orthosis at any time by repeating Step 2. To achieve an optimal therapy result you should wear the orthosis over your socks while wearing runners/shoes.

## Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the orthosis can only be realised if it is applied correctly. The orthosis is largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may negatively impact on the durability of the product. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the orthosis is not suitable for treating more than one patient. The welded-on label on the foot base is important for the identification and traceability of the product. We therefore ask you not to remove the label under any circumstances.

## Washing and care instructions

Please observe the care instructions on the foot base of the orthosis. Wash the orthosis separately the first time (the colour might come out). For improved cleaning results, we recommend that you remove the straps completely. Handwash the orthosis and straps with laundry detergent for delicate fabrics every day or between uses at approx. 30 °C. We recommend using the mild Juzo Special Detergent for this. Afterwards, please rinse out the orthosis well and allow it to dry at room temperature. To reduce the drying time, you can place the orthosis on a thick terry towel and pat it dry. Do not leave in the towel and do not place on a heater or in the sun to dry. This Juzo product must not be cleaned chemically. The orthosis is not suitable for tumble drying.

## Material composition

For full details, please refer to the label on the foot base of the orthosis.

## Storage information and usage duration

Store in a dry place and protect from sunlight. The medical retailer or doctor prescribing the orthosis will advise on exactly how long it can be used for.

## Notes for the orthopaedic technician

The orthosis can be adjusted to suit the width of the foot. To do so, release the velcro fastener at the base and adjust the separation between the shells as required.

## Indications

Conservative and post-operative treatment of ligament injuries, chronic instability of the ankle joint, following ligament lesions and/or distortions, stabilisation of the ankle joint during sport.

## Contraindications

If the following symptoms exist, the orthosis should only be worn after consultation with the physician: lymph drainage disorders, arterial flow impairment, post-traumatic conditions, skin disorders or skin irritations; Wounds in the treatment area are to be covered with sterile dressings.

The Julius Zorn Group assumes no liability as a result of any contraindicated use of this product.

## Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your symptoms worsen during the time you are wearing the product, please take off the orthosis and consult a medical specialist immediately. The manufacturer is not liable for damages/injuries which have been caused by improper handling or misuse.

In the event of complaints related to this product, such as damage to the material or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

## Disposal

As a rule, the product is to be destroyed via the conventional disposal route after the end of use. Please observe the applicable local/national regulations.

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**Julius Zorn GmbH** · Juliusplatz 1 · 86551 Aichach · Germany · [juzo.com](http://juzo.com)
**Berro AG** · Austrasse 7 · 9055 Bühler · Suisse
**UK responsible person: Juzo UK Ltd**