





JuzoPro

Lumbal Xtec Strong / Plus







Lumbalorthesen · Lumbar orthosis

Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning · Käyttöohje · Bruksanvisning · Οδηγίες χρήσης · Kullanım talimatları · Upute za uporabu · Instrukcja użycia · Инструкција по експлуатации · Návod na používanie · Navodila za uporabo · تعليمات الاستخدام

JuzoPro Lumbal Xtec Strong

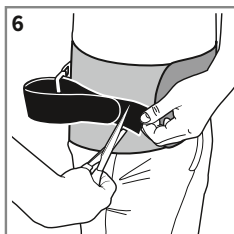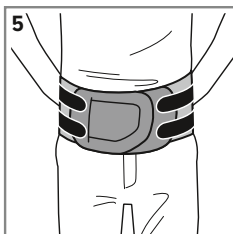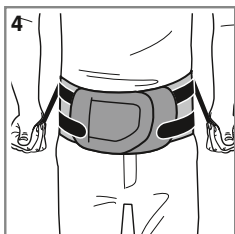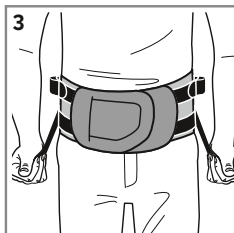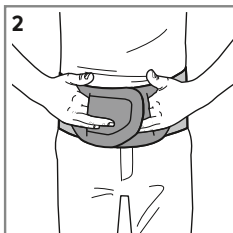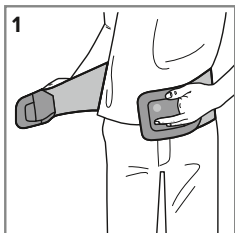JuzoPro Lumbal Xtec Plus

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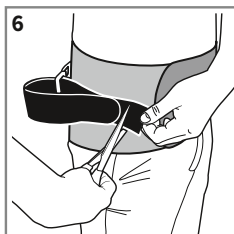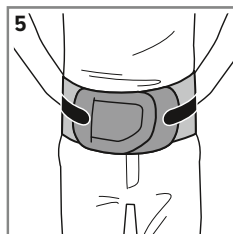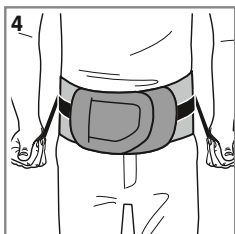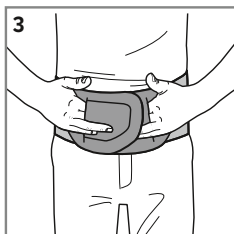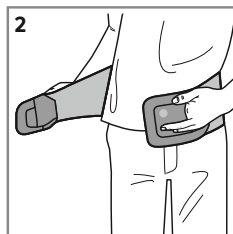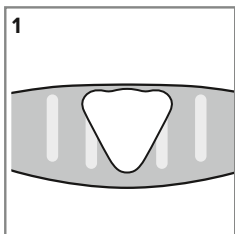EN

Purpose

Thanks to its compressive knitted fabric, in combination with stabilising rigid elements and optional straps, the JuzoPro Lumbal Xtec Strong / JuzoPro Lumbal Xtec Plus orthosis supports the lumbar region of the back, thus relieving it during movement.

JuzoPro Lumbal Xtec Plus: The sacral pad provides stabilisation in the end range of motion and, thanks to friction nubs, has a massaging effect on the muscles.

Here's how to put on your

JuzoPro Lumbal Xtec Strong correctly:

1. Before applying the orthosis please loosen the pair of straps on each side of the fastening area to ensure the orthosis is free of tension.
2. To make the product easier to put on, please use two fingers to reach into the finger pockets attached to the closure sections. Now place the orthosis around your body (Fig. 1) and close the velcro fastener on the front of the orthosis in the following order: First place the left part of the fastener on your stomach (Fig. 1) and then place the right side of the fastener tightly across it and fasten it (Fig. 2).

- Now detach the lower two pulling straps and simultaneously pull these away from the body (Fig. 3). You can use these to set the optimum tensioning action of the orthosis for your personal requirements and pain sensation. Next secure the ends of the straps anywhere on the velcro fastening area. Repeat the same steps for the upper two straps (Fig. 4 + 5).
- You can individually adjust the length of the straps by cutting them above where they attach to the fastening area (Fig. 6).

Here is how to put on your

JuzoPro Lumbal Xtec Plus correctly:

- Fasten the stabilising girdle as described by your physician or the orthopaedic technician between the two rear bars on the fleece belt and press together firmly. The tip of the stabilising girdle must point downwards (Fig. 1).
- Before applying the orthosis please loosen the two straps to ensure the orthosis is free of tension.
- To make the product easier to put on, please use two fingers to reach into the finger pockets attached to the closure sections. Now place the JuzoPro Lumbal Xtec Plus around your body (Fig. 2) and close the touch fasteners on the front of the orthosis in the following order: First place the left part of the fastener on your stomach (Fig. 2) and then place the right side of the fastener tightly across it and fasten it (Fig. 3).
- Now detach the two pulling straps and simultaneously pull these downwards and away from the body at a slight angle (Fig. 4). You can use these to set the optimum tensioning action of the orthosis for your personal requirements and pain sensation. Next secure the ends of the straps anywhere on the velcro fastening area (Fig. 5).
- You can individually adjust the length of the straps by cutting them above where they attach to the fastening area (Fig. 6).

The orthosis is properly in place when it covers the abdomen without exerting too much (painful) pressure. The orthosis must be put on, or must be pulled down, so that the lower edge of the orthosis is situated just above the groin. You can re-adjust it at any time if necessary.

Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the orthosis can only be realised if it is applied correctly. The orthosis is largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may negatively impact on the durability of the fabric. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please

consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the orthosis is not suitable for treating more than one patient. The sewn-in textile label is important for the identification and traceability of the product. We therefore ask you not to remove the label under any circumstances.

Washing and care instructions

Please observe the care instructions on the textile label attached to your orthosis. Take the rear stabilising girdle out of the orthosis, close the touch fasteners and wash the orthosis in gentle or delicate cycle at approx. 30 °C. Wash the orthosis separately the first time (possibility of colour bleeding). If washing by hand, rinse the orthosis well and allow it to dry at room temperature. If possible, lay flat. Do not wring it out. We recommend using the mild Juzo Special Detergent. To reduce the drying time, you can place the orthosis on a thick terry towel, roll it up tightly and squeeze it firmly. Do not leave it in the towel and do not place it on a heater or in the sun to dry. The orthosis is not suitable for tumble drying. Please fasten the stabilising girdle back into the orthosis again when the orthosis is completely dry. This Juzo product must not be cleaned chemically. You can wipe the stabilising girdle clean with a damp cloth. The cover can be washed just like the orthosis.

Material composition

Details are provided on the label sewn into your orthosis.

Notes for the orthopaedic technician

The stabilisation rods can be adjusted to the individual requirements of the patient.

JuzoPro Lumbal Xtec Strong: To shape the rods or, depending on the indication, remove them completely, simply take them out of the pockets.

JuzoPro Lumbal Xtec Plus: The orthosis can be dismantled simply by detaching the pad and strap element.

Storage information and usage duration

Store in a dry place and protect from sunlight. The medical retailer or doctor prescribing the orthosis will advise on exactly how long it can be used for.

Indications

JuzoPro Lumbal Xtec Strong:

Osteoarthritis of the facet joint (joint wear), low back pain/lumbosciatica (severe pain and irritation in the lumbar region), intervertebral disc protrusion, herniated disc (painful bulging of the intervertebral discs in the lumbar region), spondylolysis (displacement of the connection between two vertebrae), degenerative spondylolisthesis/Grade I with low back pain (degenerative vertebral slippage), myostatic, muscular insufficiency (postural weakness), lumbago/with SI joint involvement (suddenly occurring pain with subsequent limitation of movement), lumbar spinal canal stenosis (narrowing of the spinal canal).

JuzoPro Lumbal Xtec Plus:

Chronic, recurring low back pain/ligamentoses (severe persistent/recurring pain in the lumbar region), osteochondrosis/spondylosis/spondylarthrosis (joint wear), lumbosciatica/radiculopathy (mild pain and irritation in the lumbar region), spondylolysis (displacement of the connection between two vertebrae).

Contraindications

In case of the following syndromes or clinical pictures, the orthosis should only be worn after prior consultation with the treating physician:

Skin diseases or skin irritations;

Wounds in the treatment area are to be covered with sterile dressings.

The Julius Zorn GmbH Group assumes no liability as a result of any contraindicated use of this product.

Side effects

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritations) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your medical condition deteriorates while the use of our products, please go to your physician immediately. The manufacturer is not liable for damages/injuries which have been caused by improper handling or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

As a rule, the product is to be destroyed via the conventional disposal route after the end of use. Please observe the applicable local/national regulations.

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Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany · juzo.com
CH REP Berro AG · Austrasse 7 · 9055 Bühler · Suisse
UK responsible person: Juzo UK Ltd