JuzoPro

## Lumbal Xtec / Lumbal Xtec Light





Lumbalorthese · Lumbar orthosis

---



Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning · Käyttöohje · Bruksanvisning · Οδηγίες χρήσης · Kullanım talimatları · Upute za uporabu · Instrukcja użycia · Инструкција по експлуатации · Návod na používanie · Navodila za uporabo · تعليمات الاستخدام

---

### JuzoPro Lumbal Xtec

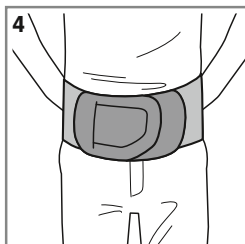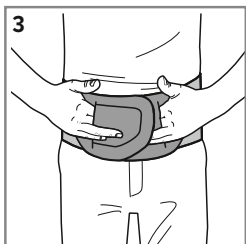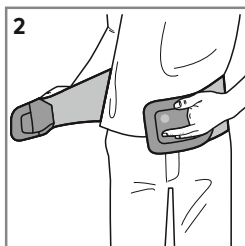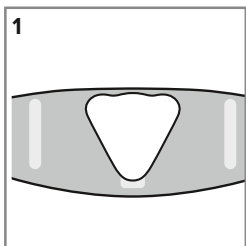### JuzoPro Lumbal Xtec Light

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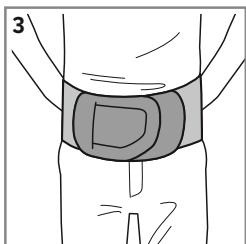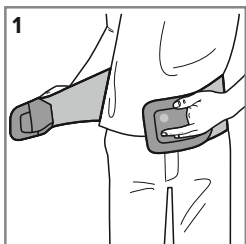EN

## Purpose

Thanks to its compressive knitted fabric, in combination with stabilising rigid elements and optional straps, the JuzoPro Lumbal Xtec/JuzoPro Lumbal Xtec Light orthosis supports the lumbar region of the back, thus relieving it during movement.

JuzoPro Lumbal Xtec: The sacral pad provides stabilisation in the end range of motion and, thanks to friction nubs, has a massaging effect on the muscles.

## Here is how to put on your

### JuzoPro Lumbal Xtec correctly:

1. Fasten the stabilising girdle as described by your physician or the orthopaedic technician between the two rear bars on the fleece belt and press together firmly. The tip of the stabilising girdle must point downwards (Fig. 1).
2. To make the product easier to put on, please use two fingers to reach into the finger pockets attached to the closure sections (Fig. 2).
3. Now place the orthosis around your body and close the touch fastener on the front of the orthosis in the following order: first position the left closure on the abdomen, then place the right closure on top of it while applying tension and attach them to each other (Fig. 3).
4. The orthosis must be put on low enough or be pulled down after donning so the lower edge of the orthosis is just above the groin (Fig. 4).

The orthosis fits properly when it surrounds the body without applying too much (painful) pressure. You can re-adjust the orthosis by repeating step 3 at any time. Caution: rough, unfiled finger nails or sharpedged rings etc. may damage the material.

## Here is how to put on your

### JuzoPro Lumbal Xtec Light correctly:

1. To make the product easier to put on, please use two fingers to reach into the finger pockets attached to the closure sections (Fig. 1).
2. Now place the orthosis around your body and close the touch fastener on the front of the orthosis in the following order: First position the left closure on the abdomen, then place the right closure on top of it while applying tension and attach them to each other (Fig. 2).

**3.** The orthosis must be put on low enough or be pulled down after donning so the lower edge of the orthosis is just above the groin (Fig. 3).

The orthosis fits properly when it surrounds the body without applying too much (painful) pressure. You can re-adjust the orthosis by repeating step 2 at any time. Caution: rough, unfiled finger nails or sharp-edged rings etc. may damage the material.

#### **Please note:**

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The effectiveness of the orthosis can only be realised if it is applied correctly. The orthosis is largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may negatively impact on the durability of the fabric. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the orthosis is not suitable for treating more than one patient. The sewn-in textile label is important for the identification and traceability of the product. We therefore ask you not to remove the label under any circumstances.

#### **Washing and care instructions**

Please observe the care instructions on the textile label attached to your orthosis. Please remove the back pad from the orthosis, close the touch fastener and wash the orthosis every day or between uses on a gentle or delicate wash cycle at approx. 30 °C. Wash the orthosis separately the first time (possibility of colour bleeding). If washing by hand, rinse the orthosis well and allow it to dry at room temperature. If possible, lay flat. Do not wring it out. We recommend using the mild Juzo Special Detergent. To reduce the drying time, you can place the orthosis on a thick terry towel, roll it up tightly and squeeze it firmly. Do not leave it in the towel and do not place it on a heater or in the sun to dry. The orthosis is not suitable for tumble drying. Please fasten the stabilising girdle back into the orthosis again when the orthosis is completely dry. This Juzo product must not be cleaned chemically. You can wipe the stabilising girdle clean with a damp cloth. The cover can be washed just like the orthosis. Tip: a mesh laundry bag provides the knitted fabric with additional protection.

#### **Materials used**

Details are provided on the label sewn into your orthosis.

#### **Storage information and usage duration**

Store in a dry place and protect from sunlight. The medical retailer or doctor prescribing the orthosis will advise on exactly how long it can be used for.

## **Indications**

#### **JuzoPro Lumbal Xtec:**

Chronic low back pain/ligamentoses (ongoing pain in the lumbar region), degenerative spondylolisthesis (degenerative vertebral slippage), myostatic, muscular insufficiency (postural weakness), osteochondrosis/spondylosis/spondylarthrosis (mild joint wear), post-discectomy syndrome (pain syndrome after surgical removal of intervertebral disc tissue), SI joint syndrome.

#### **JuzoPro Lumbal Xtec Light:**

Chronic low back pain/ligamentoses (ongoing pain in the lumbar region), myostatic, muscular insufficiency (postural weakness), spondylolisthesis, post-discectomy syndrome (pain syndrome after surgical removal of intervertebral disc tissue), SI joint syndrome.

#### **Contraindications**

In case of the following syndromes or clinical pictures, the orthosis should only be worn after prior consultation with the treating physician: skin diseases or skin irritation; Wounds in the treatment area are to be covered with sterile dressings. The Julius Zorn GmbH Group assumes no liability as a result of any contraindicated use of this product.

#### **Side effects**

There are no known side effects when used as directed. However, if negative alterations (such as for example skin irritation) should occur while the use of our products has been prescribed, please go to your physician or your specialist dealer immediately. If an incompatibility should be known against one or several ingredients of this product, please check back with your physician before use. If your symptoms worsen during the time you are wearing the product, please take off the orthosis and consult a medical specialist immediately. The manufacturer is not liable for damages/injuries which have been caused by improper handling or misuse.

In the event of complaints related to this product, such as damage to the material or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

#### **Disposal**

As a rule, the product is to be destroyed via the conventional disposal route after the end of use. Please observe the applicable local/national regulations.

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**Julius Zorn GmbH** · Juliusplatz 1 · 86551 Aichach · Germany · [juzo.com](http://juzo.com)
**CH REP Berro AG** · Austrasse 7 · 9055 Bühler · Suisse
**UK responsible person: Juzo UK Ltd**