

easy-slide

another Arion brand

Easy-Slide Arm®

The ultimate donning aid for arm sleeves



Problem definition

For both caregivers and patients, the problem of donning arm sleeves is one which occurs on a daily basis.

For caregivers, donning and removing arm sleeves is a major source of physical strain. Even patients who are able to carry out the operation independently may find the process an increasingly arduous one. This often has a direct effect on therapy compliance.

Solution

Easy-Slide Arm has been specifically designed to alleviate the problems experienced when donning arm sleeves. Easy-Slide Arm is made from very smooth material, which has been treated with a special coating. These qualities create a super-smooth aid that reduces friction significantly.

The friction reduction of Easy-Slide Arm helps minimize the physical strain otherwise experienced by caregivers and patients. In turn, this has a positive effect on the therapy compliance of patients. Also, with clear and simple instructions-for-use, patients can enjoy the freedom and convenience of independently donning arm sleeves for a long time.

Users

Easy-Slide Arm provides an effective solution for patients who wear arm sleeves. Thanks to Easy-Slide Arm, hosiery can now be donned without difficulty.

Furthermore, Easy-Slide Arm is an essential aid for caregivers who are faced with the prospect of donning their patients' arm sleeves on a daily basis.

Easy-Slide® Arm

The attractive alternative

Product benefits

- Reduction of physical strain for both caregiver and user
- Increase in patient's independence and therapy compliance
- Stockings retain therapeutic graduated compression longer
- Easy to use
- Saves time and money
- Durable
- Take it anywhere
- Custom-made products available

Product range

Art. No.	Size	Length
E01505	arm medium	55,5 cm
E01506	arm large	62,5 cm




Slide the Easy-Slide over the arm



Slide the stocking over the Easy-Slide



Hook the loop on a door handle, secure the arm sleeve with your free hand

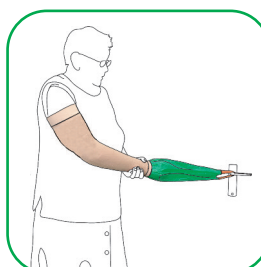

Grasp the loop on the Easy-Slide