



the slide solution

simply smile

English

ARION
simply smile

Introduction

Putting on and taking off therapeutic compression stockings and tights is often a difficult task which recurs on a daily basis. To perform these procedures more easily and with less effort Arion developed various application aid products. This product range is summarised under the name The Slide Solution.

The Slide Solution offers the total solution: all types putting on and taking off applications, both in single version as well as in a +package. Thus, offering a suitable product for each compression stocking wearer. The individual products are described in this brochure.

Indispensable for putting on and removing compression stocking units with open and closed toe are The Slide Solution Gloves. For pulling on arm stockings and above-knee prostheses Arion respectively developed Easy-Slide Arm and Proth-Pro®.

Problem definition

For both caregivers and patients the task of putting on compression stockings and tights is recurring on a daily basis. For caregivers, this procedure is a major source of physical strain. Even patients who are able to put on and take off compression stockings independently find this process an increasingly arduous one. This often has a direct effect on therapy compliance.

Solution

Our Slide Solution products have been specifically designed to alleviate the problems that are encountered when putting on and taking off compression stockings and tights with open or closed toe. The Slide Solution products are made from smooth material, which has been treated with a special coating. These properties create an ultra-smooth aid that reduces friction significantly. The reduced friction minimises the physical strain otherwise experienced by caregivers and patients. In turn, this has a positive effect on the therapy compliance of patients. The clear and simple instructions for use allow patients to enjoy the freedom and convenience of independently putting on compression hosiery for a long time.

Target group

1. Patients who wear compression stocking and tights, but have great difficulties with putting on and taking off and want to stay independent.
2. Caregivers who are faced with the prospect of putting on their patients' compression hosiery on a daily basis.

Product benefits

- Reduction of physical strain for both caregiver and patient
- Increase in patient's independence and therapy compliance
- Stockings retain therapeutic graduated compression longer
- Easy to use
- Saves time and money
- Durable
- Take it anywhere
- Also custom made available

Quality and patents

The price of quality

Quality and user experience are key drivers for the development of our products. The Magnide® is a good example of this. Although it looks like a very simple product at first sight, the strength of the product can be found in the details. Arion's aim was to develop the world's simplest and best aid for putting on compression stockings with closed toe. All of this in combination with the highest quality and strong worldwide patents.

Arion searched for the best of materials; magnets, fabric, yarn, etcetera. The fabric is the base of all our Slide Solution products, a very smooth cloth which has been treated with a special coating. Together with the double-layered application this quality creates an ultra-smooth aid that reduces friction significantly. The magnets need to have the perfect magnetic strength; not too strong and not too weak, as the Magnide® needs to stay closed whilst putting on the stocking, but it should be easy to open when removing from the stocking. Moreover, thickness and size are particularly important. After extensive testing of magnets at Arion, together with TUV, the perfect magnet was found. In order to keep the magnets firmly in place, they have been put into "pockets". To prevent the magnets from slipping, the material of the pockets has a slightly rough feel.

Finally all components have been thoroughly tested and researched to maximise quality.



The value of patents

A patent is an exclusive right intended to protect a new product, method or technology. Arion received 7 international patents for its Slide Solution products over the last few years. Because of this protection, we are able to continue developing innovative products. Innovation remains the main driver of Arion!



Review Magnide



'Personally, I find the magnet closing system a significant improvement compared to the Easy-Slide Caran. The reasons for this: Magnide® opens easier and the small chance of damaging the stocking is no longer present. Less force is needed and the folding is less complicated. I have had the product tested by various customers during several weeks. Especially the improved user-friendliness was seen as very positive. To summarise, I would definitely recommend the Magnide® because putting on your compression stockings becomes easier yet again!'

Onno Smeets, Advisor compression stockings, Spronken Orthopedie

Portfolio



Easy-Slide



Application aid for open toe compression stockings and tights.



Easy-Slide Arm



Application aid for arm stockings.



Easy-Off



Application aid for pulling off open or closed toe compression stockings and tights.



Sim-Slide®



The ultimate application aid for putting on and pulling off open toe compression stockings and tights in one product.



Magnide®



The ultimate application aid for closed toe compression stockings and tights



Magnide® on/off



The ultimate application aid for putting on and taking off closed toe compression stockings and tights in one product.



The Slide Solution Gloves



Indispensable tool for putting on and pulling off compression stockings in combination with The Slide Solution products.



SlideX®




The additional application aid for putting on and taking off compression stockings and tights by caregivers to use in combination with Easy-Slide, Sim-Slide and Magnide.



Proth-Pro®



The improved application aid for putting on an above-knee prosthesis with suction socket.

	Easy-Slide	Sim-Slide	Magnide					
								
PUTTING ON / DONNING								
Open toe	●	●	○					
Closed toe			●					
Arm stocking								
TAKING OFF / DOFFING								
Open toe		●						
Closed toe		○						
PRODUCT BENEFITS								
Very smooth material with special coating	●	●	●					
Double layer application	●	●	○					
Reducing physical strain patient	●	●	●					
Reducing physical strain caregiver	●	●	●					
Increased autonomy and therapy compliance	●	●	●					
Easy to use	●	●	●					
Easy to carry	●	●	●					
Easy to fix by elastic band around the ankle		●						
Very user-friendly by magnetic closure			●					
Easy putting on and taking off by optimal grip								
Prevents damage to the stocking by nails and jewelry								
MAINTENANCE								
Hand wash at low temperature of 30°C	●	●	●					
Do not dry clean	●	●	●					
Dry with a towel (never iron or tumble)	●	●	●					
WARRANTY	6 months	6 months	6 months					
SIZE OVERVIEW (WITH ARTICLE NUMBERS)								
Easy-Slide	Art. number	Size	Shoe size	Sim-Slide	Art. number	Size	Shoe size	Magnide
	E01500	Extra Small	≤ 33		EOSL01	Small	34-37	
	E01501	Small	34-37		EOSL02	Medium	38-41	
	E01502	Medium	38-41		EOSL03	Large	42-45	
	E01503	Large	42-45		EOSL04	Extra Large	≥ 46	
E01504	Extra Large	≥ 46						
Easy-Off	Art. number	Size	Shoe size	Easy-Slide arm	Art. number	Size	Length	Easy+ Magnide
	E01510	Small	34-37		E01505	Medium	55,5cm	
	E01511	Medium	38-41		E01506	Large	62,5 cm	
	E01512	Large	42-45					

CONTACT DETAILS



Magnide on/off SlideX * Easy-Off Easy-Slide arm Easy+ Easy-Slide Easy+ Magnide Gloves

○ ●	● ●				●	● ● ●
○ ●	● ●	● ●				● ● ●
● ○ ● ● ● ● ● ● ● ●	● ● ● ● ● ●	● ● ● ● ● ● ● ●	● ● ● ● ● ● ● ●	● ● ● ● ● ● ● ●	● ○ ● ● ● ● ● ● ● ●	
						● ●
● ● ●	● ●	● ● ●	● ● ●	● ● ●	● ● ●	
6 months	6 months	6 months	6 months	6 months	6 months	

Art. number	Shoe size	
Medium	≤ 40	 XL also suitable for extra wide feet with A upto 28 cm
Large	41-45	
Extra large	≥ 46	

Magnide on/off

Art. number	Size	Shoe size
Medium	≤ 40	 XL also suitable for extra wide feet with A upto 28 cm
Large	41-45	
Extra large	≥ 46	

SlideX

Art. number	Size	Shoe size
E01520	Medium	< 20 cm
E01521	Large	> 20 cm


Art. number	Size	Shoe size
E01610	Small	34-37
E01611	Medium	38-41
E01612	Large	42-45
E01613	Extra Large	≥ 46

Easy+ Magnide

Art. number	Size	Shoe size
E01619	Medium	≤ 40
E01620	Large	≥ 41

Gloves

Art. number	Size	Shoe size
E08030	Small	≤ 6,5
E08010	Medium	7 - 8,5
E08020	Large	≥ 9



*only applicable in combination with Easy-Slide,
* Sim-Slide, Magnide, Magnide on/off and Easy-Off





Toomac have an extensive range of solutions for venous and lymphatic conditions. Proud suppliers of Juzo medical compression and lymphedema garments in all sizes, styles and grades of compression, we also offer solutions for donning and doffing of compression garments. Our focus is on bringing the latest product innovations and developments from world class manufacturers to the New Zealand market, in conjunction with our customers, for the benefit of their patients. Strong business partnerships with International Suppliers coupled with a commitment to provide quality service and clinically reputable products to our customers, ensures patients are able to receive the best available solutions.

Toomac Solutions

32C Poland Rd
Glenfield, Auckland
Ph 0800 TOOMAC (866622)
www.toomac.co.nz
sales@toomac.co.nz