




On + Off




The 2in1 donning and doffing aid for closed-toe medical compression stockings and tights with a unique magnetic system

Arion Magnide® 2in1 offers a smooth transition of the compression garment onto the patient's skin thanks to its unique sliding properties. It makes the application of compression garments safe and easy, minimizing their wear and tear. Arion Magnide® 2in1 has a flexible sizing to fit different sizes of limbs. This is particularly important for patients diagnosed with lower limb edema (swelling). Arion Magnide® 2in1 is a simple solution for healthcare professionals to recommend and for compression wearers to use in combination with medical compression therapy.

Before you start:






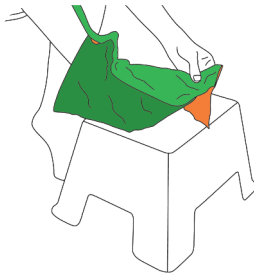


Important!
Don't push the foot to the
very end of the Magnide®,
leave some free space
at the end






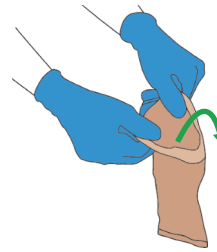


In case the compression
garment has a silicon border,
make sure to turn it
outwards

Donning instructions

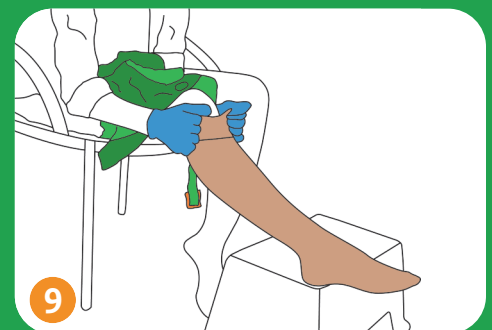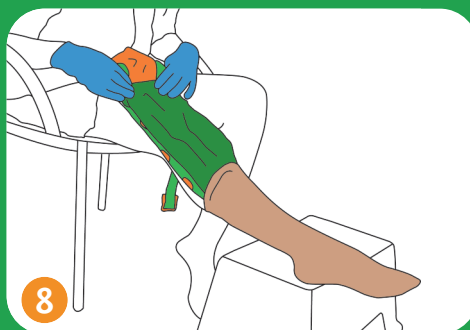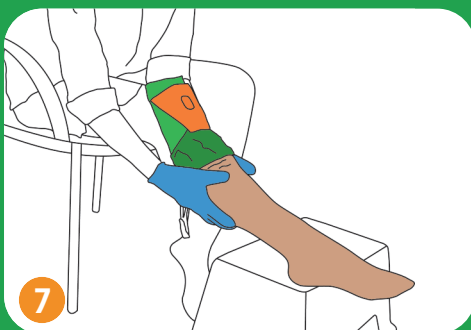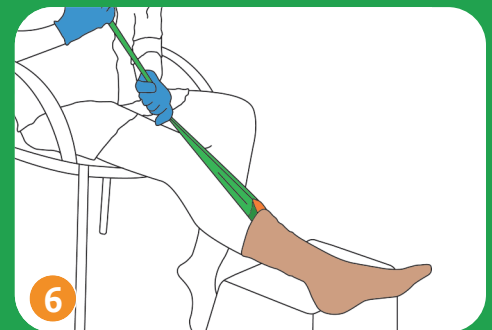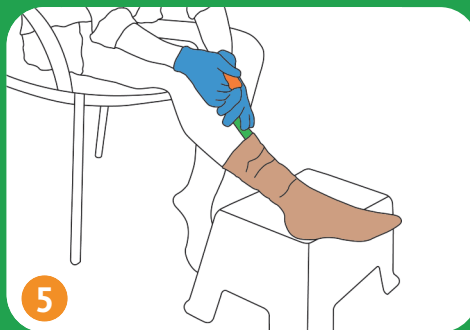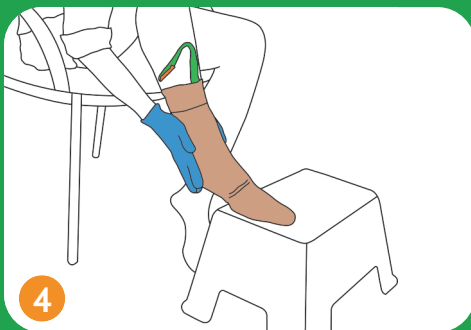


Doffing instructions

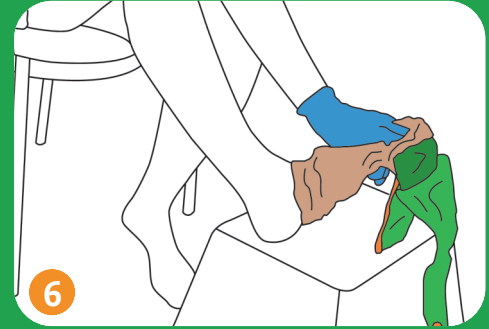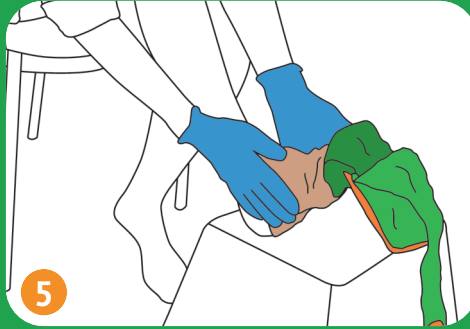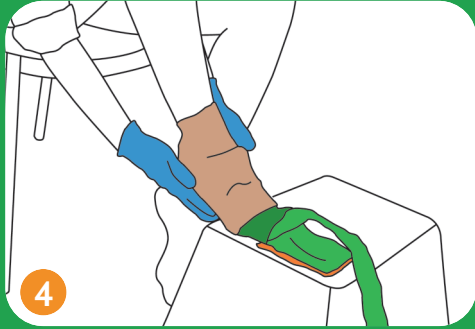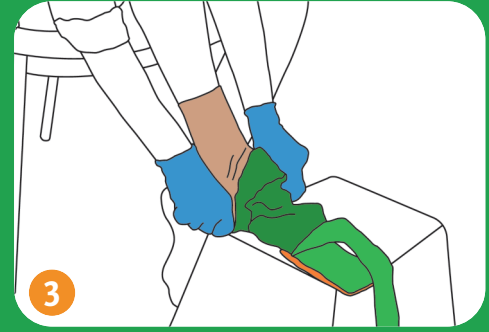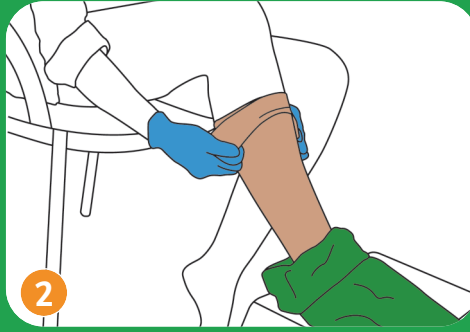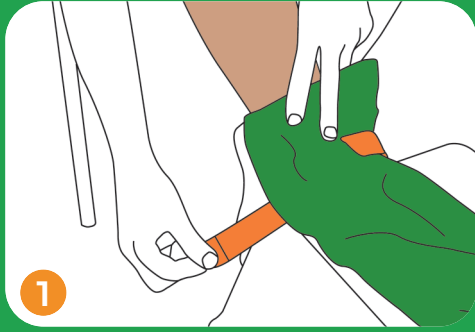


Fix the top of the loop to the top of the Magnide® 2in1 by using the magnet




Turn over the Magnide® 2in1 and place it on a flat surface

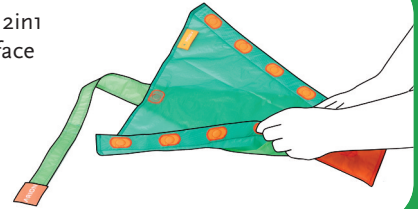

Fold both green sides to the middle. The orange magnet tabs must meet each other and make contact.

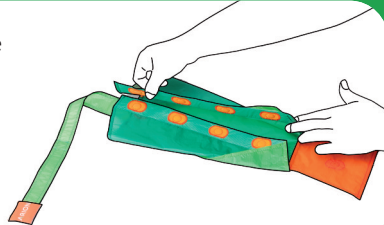

Close the orange flap (near the toes) over the Magnide® 2in1 so that the magnets make contact

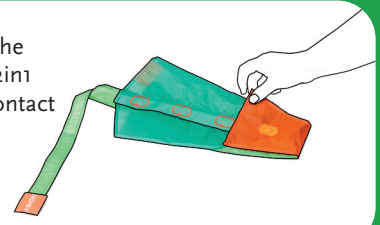

Preparation for re-use

Compression therapy made easier

Arion donning and doffing aids for medical compression garments

Easy to Use

- Minimize friction between compression garment and skin
- Reduce physical strain for both compression wearers and caregivers
- Promote independence of compression wearers

Safe to Apply

- Reduce risk of skin damage ; can be especially beneficial in case of high skin sensitivity or leg ulcers
- Contribute to a longer garment lifetime by preventing damage from fingers and nails
- Help maintaining the original compression longer and avoiding garments from overstretching and distortion

Simple Solutions

- Application aids for all types of compression garments: stockings, tights, arm sleeves, and gloves
- Designed for different shapes and sizes of limbs
- Support the concordance to compression therapy

Arion Magnide® 2in1

SIZES

Art. Nr.	Size	Shoe Size EU	Shoe Size (UK)	
			Woman	Man
E01540	Medium	≤ 38	≤ 5,5	≤ 6
E01541	Large	39 - 43	6 - 8	6,5 - 9
E01542	X-Large	≥ 44	≥ 9	≥ 9,5

XL also suitable for extra wide feet (circumference up to 11"/28 cm)



Should you need another size, Arion can customize the product according to your needs.

Toomac Solutions
DIVISION OF TOOMAC HOLDINGS LTD

Distributed by:

Toomac Solutions
32D Poland Road • Glenfield, Auckland
Ph 0800 TOOMAC (866622)
www.toomac.co.nz • sales@toomac.co.nz

ARION
SLIDE SOLUTIONS

Manufactured by:

Arion Slide Solutions B.V.
Rutherford 11 • 6422 RE Heerlen • NL
www.slide-solutions.com