




## magnide

another Arion brand



Place the Magnide across the foot. Ensure that the orange flap near the toes is well folded back, so the magnet makes contact at the bottom.

1

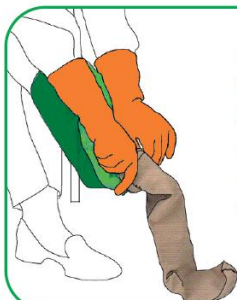


Pull the stocking over the Magnide.  
Advice: don't roll up the stocking because that makes it even more difficult to put on!

2




Slide the stocking over the Magnide until the heel of the stocking is in the right position.

3

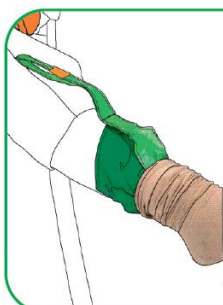

Make sure that the light green loop sticks out above the stocking so it can be grabbed.

4

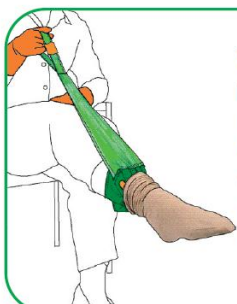


Wriggle your toes. Pull the light green loop upwards to remove the Magnide from the stocking. Regrab if possible when removing the Magnide.

5

**magnide**  
another Arion brand

Lay the Magnide flat with the inside (the side where the size label is attached) down and click the green light loop on the bottom.

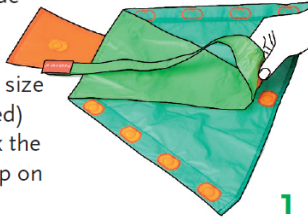


1

Turn the Magnide with the orange flap towards you and the light green loop upwards and fold the left half inside.




2

Fold the right half of the Magnide also in and click it onto the magnets of the left half.

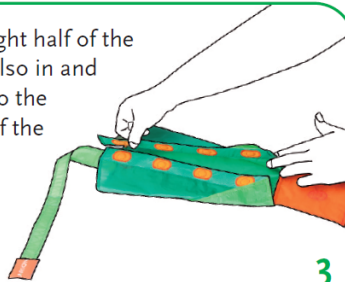

3

Finally, click the orange flap upwards and the fixed Magnide is ready for use.

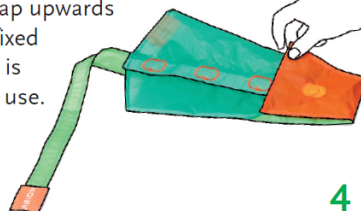

4

