

Instructions for Use and Care of your Juzo ACS Light Thigh and Knee Wraps

Important:


If there are open wounds in the area, the affected area must be covered with an appropriate wound dressing or bandage before the adjustable compression system can be applied.

Juzo ACS Light Thigh wrap (Product code JZ704).

Juzo ACS Light Thigh wraps are right and left leg specific.


1. Attach the supplied touch fasteners halfway on the tab ends so that you can secure the tabs from the outside.

2. The sewn-in adhesive border needs to be at the top of your thigh. Put your leg into the integrated lining (this is optional).

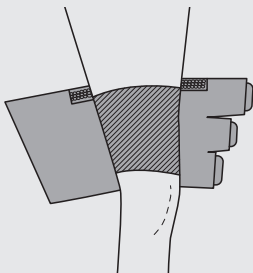


3. First, close the top tab to secure the wrap at groin level.


4. Continue closing the other tabs from the bottom to the top. Tighten them around the leg using the same tension on each tab and fix them with the touch fasteners.

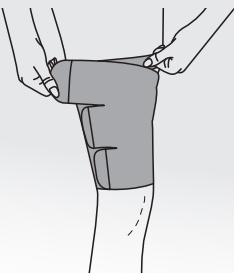

5. The supplied extender can be used as an addition for over height at the front of the leg or the side. This is optional.

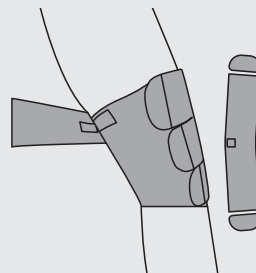


6. Check that the product exerts pleasant, clearly noticeable compression. If you feel pain or other problems, loosen the touch fasteners.

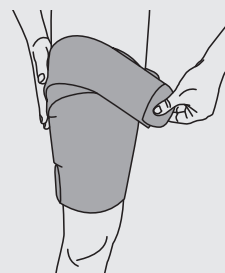


Juzo ACS Light Knee Wrap (Product code JZ703)

Important: If wearing in combination with thigh and or calf wraps, put the knee wrap on last. Juzo ACS Light knee wraps can be worn on either right or left leg.


1. Attach the supplied touch fasteners halfway on the tab ends so that you can secure the tabs from the outside.

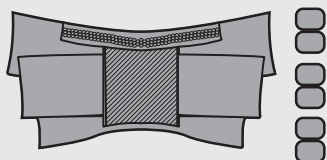


2. Put your leg inside the integrated lining (optional). Make sure that the seam is in the knee crease with cut-out opening at the back of the knee.

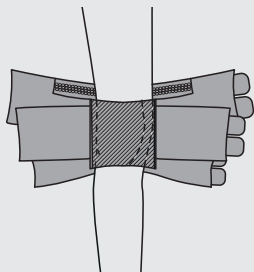

3. First, close the middle tabs to secure the wrap.

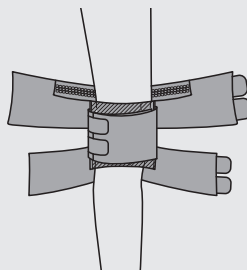

4. Tighten the lower tabs around the leg and fix them with the touch fasteners, followed by the upper tabs.

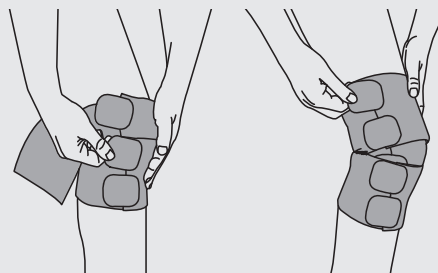

5. Check that the product exerts pleasant, clearly noticeable compression. If you feel pain or other problems, loosen the touch fasteners.

There must not be any large folds while wearing. If required, you can increase the wrap's compression by adjusting the tabs from the bottom to the top. All Juzo ACS Light products can be used with or without the incorporated lining, or with a separate liner.

Washing and care instructions

Juzo ACS Light products should be washed at least once a week or more often as required. Prior to washing remove all touch fasteners. The products can be machine-washed at 30 °C using a laundry detergent for delicate fabrics. A laundry net additionally protects the product. Please do not use fabric softener. Never use bleach. Do not tumble dry. You can reduce drying time by placing the products on a thick towel after washing, tightly rolling the towel up and firmly squeezing out excess moisture. Do not leave the product in the towel and do not dry it on a radiator or in the sun. Juzo ACS Light products must not be ironed. The products are not suitable for dry cleaning. The intervals between washing can be extended by wearing a Juzo liner. Store in a dry place and protect from sunlight.

Please refer to the detailed instructions in the Juzo Product pamphlet, enclosed in the packaging.